Nimbo

Kindergehgestell / Children Walking Aid / Kinderlooprek

Größe / Size / Maat S	- REF.: 705100100
Größe / Size / Maat M	- REF.: 705100200
Größe / Size / Maat L	- REF.: 705100300
Größe / Size / Maat XL	- REF.: 705100400

CE

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**Bedienungsanleitung
beachten**

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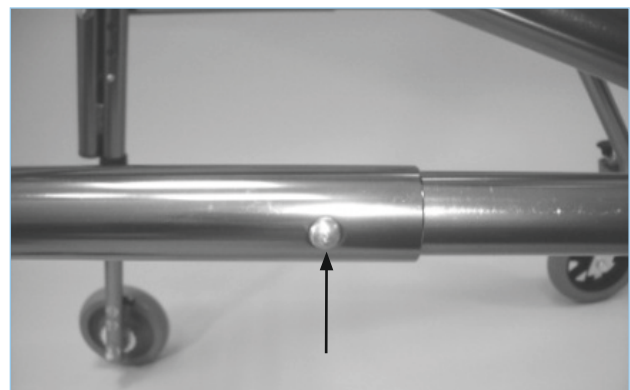
Observe instructions for use!

For indoor use only!

Use under supervision of expert staff only!

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Attention

Please read these instructions carefully before using the Kids walker Nimbo and keep it for later reference.

Please observe the following instructions to ensure properly using, safety and reliability.

Indication

Walking device for children in 4 different sizes for indoor use for compensation and improvement respectively of restricted ability of walking and for the correction of body posture problems. The reverse-roll locking devices at the rear wheels prevents from rolling backwards.

Reapplication

This product is suitable for the reapplication.

Scope of delivery contents

Unfold and assembling

The Nimbo has a foldable aluminium-pipe frame and is height adjustable in the wheels. In order to open up you lay the Nimbo on the ground, stick it possibly in the middle of the front frame and open up the frame.

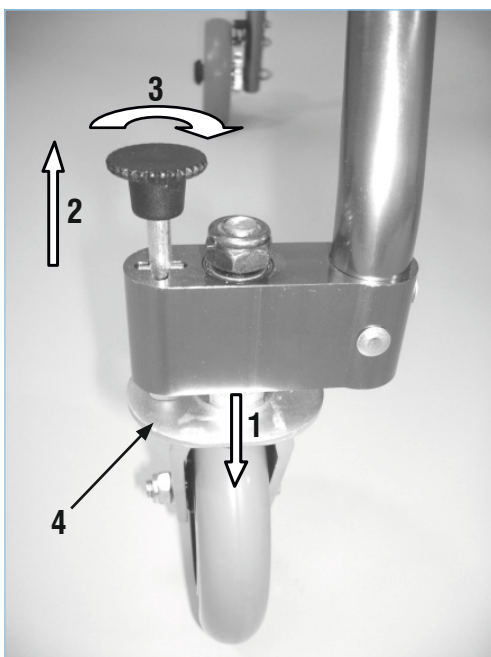
Please, note while opening that the feather clips engage in the telescope pipes always properly (Audible click!).

For assembling you must only insert the Front- and Back wheels.


Wheel lock front

In order to interlock the front wheels adjust them straight (1), then pull the Locking pin upwards (2) and turn him around 90 ° (3) until it locks in the hinged plate (4). To reset the interlock repeat the process.


Height adjustment of the wheels

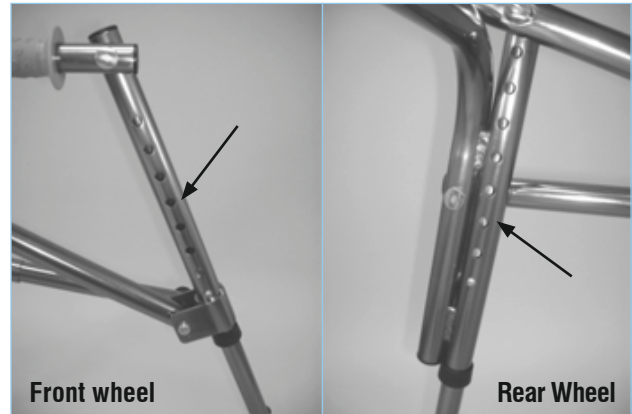The front- and the back wheels are height-adjustable in 8 steps. To adjust press the spring snap (arrow) and shift the pipe as desired. **Make sure that the spring snap always engages properly!**

Non-return lock

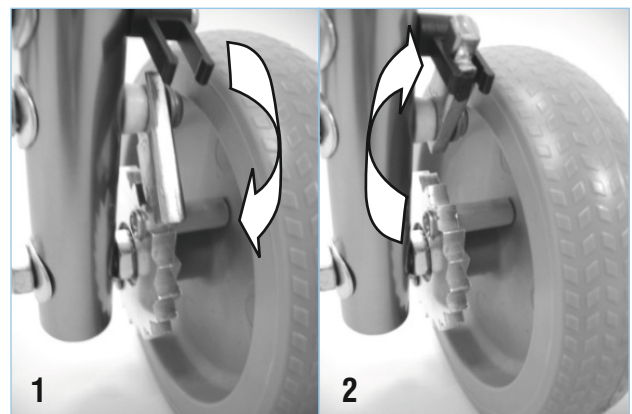
1 Non-return lock activated

2 Non-return lock deactivated

Folding

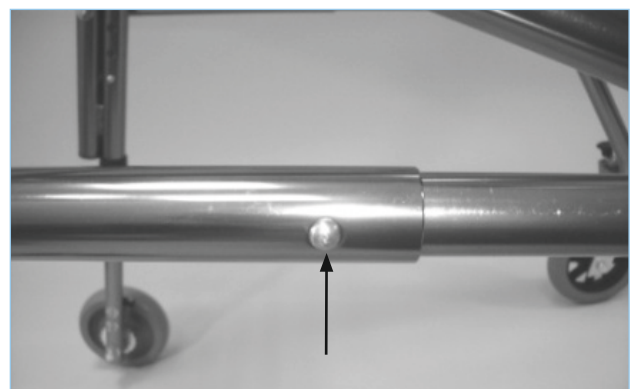To fold the kids walking aid press the spring snap (arrow) in the cross tube and fold it antipodal to the unfolding.

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